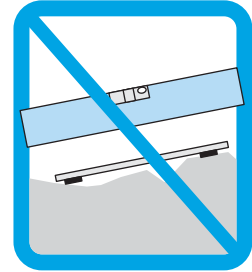
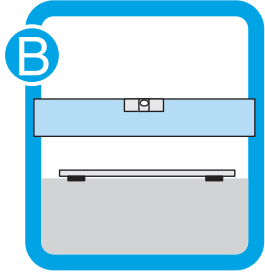
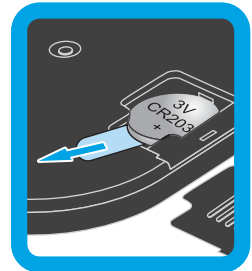
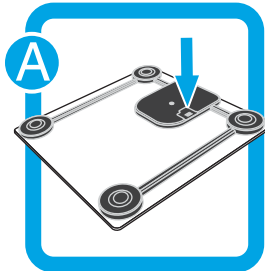
HN286 Digital Personal Scale

- ⓔⓃ Instruction Manual
- ⓔⓣ Istruzioni per l'uso
- ⓔⓔ Gebrauchsanweisung
- ⓔⓕ Mode d'emploi
- ⓔⓗ Gebruiksaanwijzing
- ⓔⓖ Manual de instrucciones
- ⓔⓞ Руководство по эксплуатации
- ⓔⓐ كتيب الإرشادات
- ⓔⓑ Наръчник с инструкции
- ⓔⓐ Návod k obsluze
- ⓔⓓ Brugsanvisning
- ⓔⓔ Εγχειρίδιο Οδηγιών
- ⓔⓕ Kasutusjuhend
- ⓔⓖ Käyttöohje
- ⓔⓗ Upute za korištenje
- ⓔⓘ Használati útmutató
- ⓔⓙ Naudojimo instrukcija
- ⓔⓚ Lietošanas instrukcija
- ⓔⓛ Bruksanvisning
- ⓔⓜ Instrukcja obsługi
- ⓔⓝ Manual de Instruções
- ⓔⓞ Manual de utilizare
- ⓔⓟ Navodila za uporabo
- ⓔⓠ Návod na obsluhu
- ⓔⓡ Instruktionsmanual
- ⓔⓓ Kullanma Kılavuzu

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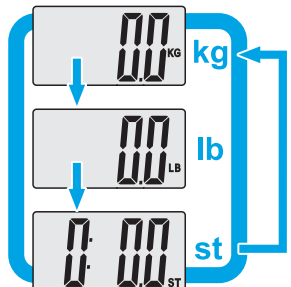
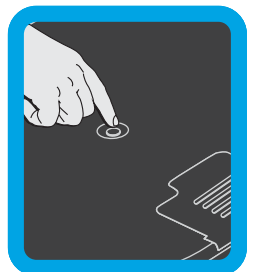
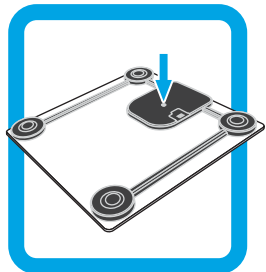
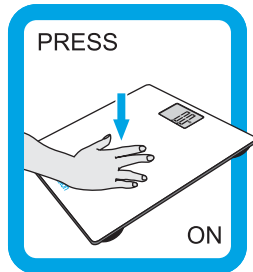
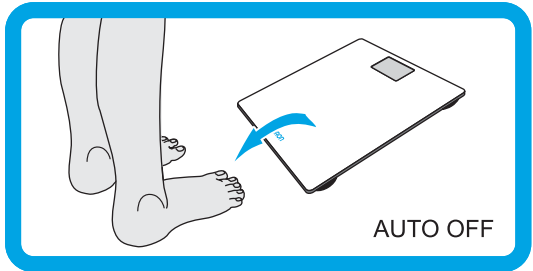
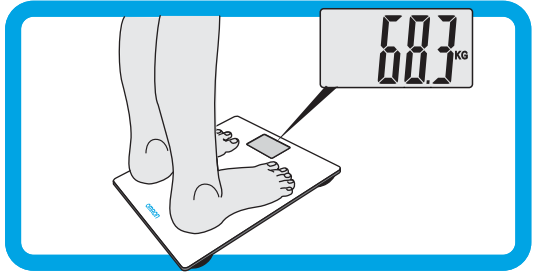
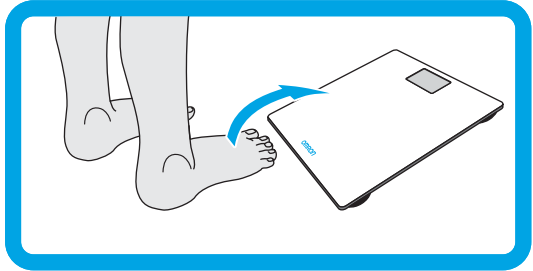
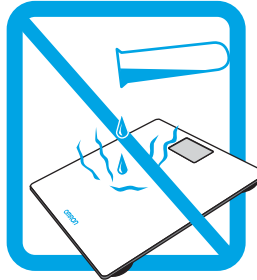
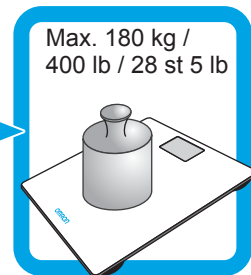
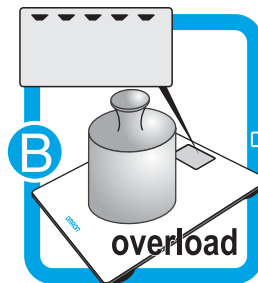
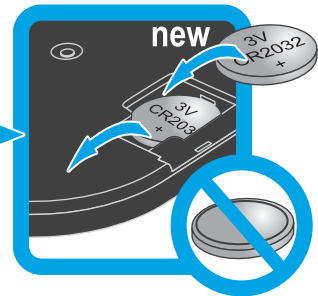
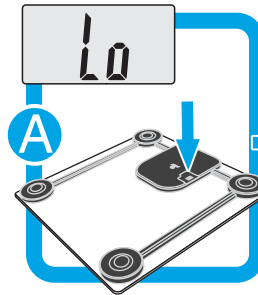
Notes on Safety

Symbols and definitions are as follows:

Warning: Improper use may result in possible death or serious injury.

Caution: Improper use may result in injury or property damage.

Warning:

- Never start weight reduction or exercise therapy without the instructions of a doctor or specialist.
- Do not use the scale on slippery surfaces, such as a wet floor.
- Keep the scale out of the reach of young children.
- Do not jump onto the scale, or hop up and down on the scale.
- Do not use this scale when your body and/or feet are wet, such as after taking a bath.
- Do not step on the edge or display area of the scale.
- People with disabilities, or who are physically frail, should always be assisted by another person when using this scale. Use a handrail or so when stepping on the scale.
- If battery fluid should get in your eyes, immediately rinse with plenty of clean water. Consult a doctor immediately.
- Do not put your fingers into any slot or inside the scale.
- Use indoors.
- Do not use for purposes other than taking weight.

Caution:

- Approved for human use only.
- The scale is slippery glass, be carefully of falling.
- Always clean the scale before use when using the unit with people with a skin disease on foot.

- Do not store in locations that exposed to chemicals or corrosive vapours.
- Please read this instruction manual carefully before use and for further information on the individual functions.
- Do not place the scale against the wall. May cause injured and malfunction.
- Do not disassemble, repair, or remodel the scale.
- Do not use battery not specified for this scale. Do not insert the battery with the polarities in the wrong direction.
- Replace worn battery with a new one immediately.
- Do not dispose of batteries in fire.
- If battery fluid should get on your skin or clothing, immediately rinse with plenty of clean water.
- Remove the battery from this scale when you are not going to use it for a long period of time (approximately three months or more).
- This scale is intended for home use only. It is not intended for use in hospitals or other medical facilities. It does not support the quality standards required for professional use.

General Advice:

- For correct measurements:
 - Please store this scale on level ground.
 - Do not place this scale on a cushioned floor surface such as on a carpet or a mat.
 - Do not store any objects on the platform.

Technical Data

Name	HN286 (HN-286-E)
Product	Digital Personal Scale
Display	0 ~ 180 kg 0.1 kg 0-400 lb: 0.2 lb 0-28 st 5 lb: 0.2 lb
Weight Accuracy	5.0 to 60.0 kg: ± 0.4 kg 60.1 to 100.0 kg: ± 0.6 kg 100.1 to 180.0 kg: ± 1%
Power Supply	One CR2032 battery (test battery is included)
Battery Life	Approximately 1 year (when scale is used for 4 measurements a day)
Operating Temperature / Humidity	+5°C to +35°C, 30% to 85% RH
Storage Temperature/ Humidity/Air Pressure	-20°C to +60°C, 10% to 95% RH, 700 hPa - 1060 hPa
Weight	Approximately 1.5 kg (battery included)
External Dimensions	Approximately 300 (W) × 18 (H) × 300 (D) mm
Package Contents	Digital Personal Scale, one CR2032 battery for test use of the scale, instruction manual

Note: Subject to technical modification without prior notice.

This device fulfills the provisions of the EMC Directive 2014/30/EU.

OMRON HEALTHCARE EUROPE B.V. guarantees this product for 2 years after date of purchase.

The guarantee does not cover battery, packaging and/or damages of any kind due to misuse (such as dropping or physical misuse) caused by the user. Claimed products will only be replaced when returned together with the original invoice / cash ticket.

Correct Disposal of This Product (Waste Electrical & Electronic Equipment)

This marking shown on the product or its literature, indicates that it should not be disposed of, with other household wastes at the end of its working life. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this product from other types of wastes and recycle it responsibly to promote the sustainable reuse of material resources.

Household users should contact either the retailer where they purchased this product, or their local government office, for details of where and how they can return this item for environmentally safe recycling. Business users should contact their supplier and check the terms and conditions of the purchase contract. This product should not be mixed with other commercial waste for disposal.

Addresses

Manufacturer 	OMRON HEALTHCARE Co., Ltd. 53, Kunotsubo, Terado-cho, Muko, KYOTO, 617-0002 JAPAN				
EU-representative <table border="1" data-bbox="106 1102 247 1161"> <tr> <td data-bbox="106 1102 176 1161">EC</td> <td data-bbox="176 1102 247 1161">REP</td> </tr> </table>	EC	REP	OMRON HEALTHCARE EUROPE B.V. Scorpius 33, 2132 LR Hoofddorp, THE NETHERLANDS www.omron-healthcare.com		
EC	REP				
<table border="1" data-bbox="210 1171 400 1251"> <tr> <td data-bbox="210 1171 400 1251">Importer in the United Kingdom</td> </tr> </table> Subsidiaries	Importer in the United Kingdom	<table border="1" data-bbox="400 1171 1005 1380"> <tr> <td data-bbox="400 1171 1005 1251"> OMRON HEALTHCARE UK LTD. Opal Drive, Fox Milne, Milton Keynes, MK15 0DG, UK www.omron-healthcare.com </td> </tr> <tr> <td data-bbox="400 1251 1005 1331"> OMRON MEDIZINTECHNIK HANDELSGESELLSCHAFT mbH www.omron-healthcare.com/distributors </td> </tr> <tr> <td data-bbox="400 1331 1005 1380"> OMRON SANTÉ FRANCE SAS www.omron-healthcare.com/distributors </td> </tr> </table>	OMRON HEALTHCARE UK LTD. Opal Drive, Fox Milne, Milton Keynes, MK15 0DG, UK www.omron-healthcare.com	OMRON MEDIZINTECHNIK HANDELSGESELLSCHAFT mbH www.omron-healthcare.com/distributors	OMRON SANTÉ FRANCE SAS www.omron-healthcare.com/distributors
Importer in the United Kingdom					
OMRON HEALTHCARE UK LTD. Opal Drive, Fox Milne, Milton Keynes, MK15 0DG, UK www.omron-healthcare.com					
OMRON MEDIZINTECHNIK HANDELSGESELLSCHAFT mbH www.omron-healthcare.com/distributors					
OMRON SANTÉ FRANCE SAS www.omron-healthcare.com/distributors					

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